

# Parallel Tape Dressing Application for Introcan Safety<sup>®</sup> 3 Closed IV Catheter

Shown with 3M<sup>™</sup> Tegaderm<sup>™</sup> IV Dressing (1633)

CORRECT APPLICATION OF THE DRESSING IS ESSENTIAL TO ENSURING THE CANNULA REMAINS SECURE.



## STEP 1

Remove the centre cut out from the frame of the dressing.  
Apply the strips to the wings of Introcan Safety<sup>®</sup> 3.



## STEP 2

Peel the paper liner from the paper frame dressing taking care not to touch the Key-Part adhesive surface. Place the dressing over the catheter so the transparent film is over the insertion site, taking care not to touch the Key-Site. Position the soft cloth tape section over the catheter wings. Overlap each side of the soft cloth tape underneath the connection between the Introcan Safety<sup>®</sup> 3 and the needlefree device.



## STEP 3

Remove the paper frame smoothing the dressing edges to increase adhesion.



## STEP 4

Loop the needlefree extension and secure this with an additional piece of clean tape. Ensure the tape strip does not cover the dressing as it may de-stabilise the dressing when the extension set is accessed.



## STEP 5

Record date, time and cannulator initials on the date strip to document dressing changes. Apply the date strip to the edge of the dressing.

**ALWAYS ENSURE CANNULA INSERTION SITE IS NOT OBSCURED**

FOR SUPPORT AND ANY QUERIES PLEASE CONTACT  
YOUR LOCAL B. BRAUN CLINICAL EDUCATION SPECIALIST OR EMAIL  
[CLINICAEDUCATIONSERVICES.BBMUK@BBRAUN.COM](mailto:CLINICAEDUCATIONSERVICES.BBMUK@BBRAUN.COM)