

Preparation



Prepare all items necessary for Diacan[®] Flex insertion, according to your organisational policies/protocols.



Select the appropriate site. Clean and disinfect the insertion site, using aseptic non-touch technique (ANTT), as per organisational policies/protocols. Apply a tourniquet if applicable.



Unpack the Diacan[®] Flex.



Flatten the wings prior to insertion. Flattening the stabilisation wings will ensure optimal fixation.



Hold the flashback chamber and remove the protective guard with a straight outward motion.



Confirm the catheter hub is properly seated on the flashback chamber.