

Ecoflac® plus

7 simple steps for use of Ecoflac® plus



TO BEGIN
Check solution with
prescription. Inspect batch no.
and expiry date of Ecoflac®
plus container and record.
Ensure clarity of infusion

solution before administration.



OPENINGTo open, grip the container and peel off one section of the covering foil. No need to disinfect the latex free port beneath the foil as this is sterile.



ADMINISTRATION
Ecoflac® plus is designed for non vented administration.
Having taken the aluminium foil off one port insert the infusion set. All major infusion sets are compatible with the port system.



PRIMING
Prime the line by inverting the
Ecoflac® plus container and
running the solution through the
infusion set. The infusion is now
ready to administer.



INTEGRATED HANGERHang the container by its integrated hanger on the drip stand.



COLLAPSIBILITY
During infusion the Ecoflac®
plus container collapses in on itself. However, the label will remain legible at all times.



DISPOSALOn completion of the infusion remove the IV set and dispose of Ecoflac® plus according to hospital policy.