



Aesculap Endoscopic Technology

DISASSEMBLY


ASSEMBLY




I Deactivate ratchet on ratchet handle
Flip down red toggle switch.
N.B.: for P0959R only




II Disconnect shaft and handle
Pull rotation knob back until it locks in place. A red ring becomes visible.
Important: Do not hold on to the movable handle component.

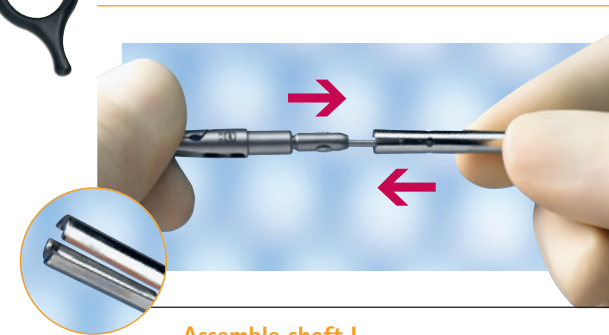


I Assemble shaft I
Slide the metal inner tube onto the jaw insert with its four spring tongues towards the working end until the tongues snap in place.

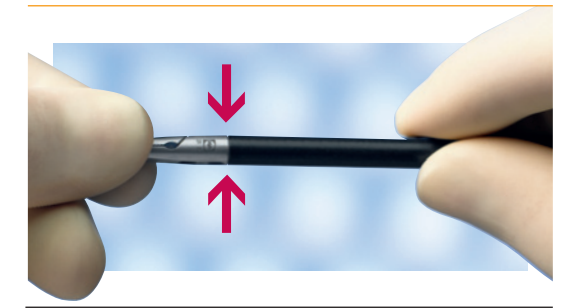


II Assemble shaft II
Slide the outer insulation tube as far as it will go over the inner metal tube.

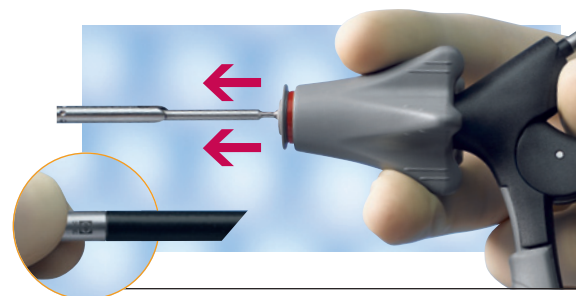


III Remove shaft from handle
Remove the shaft by pulling the instrument shaft out of the handle.

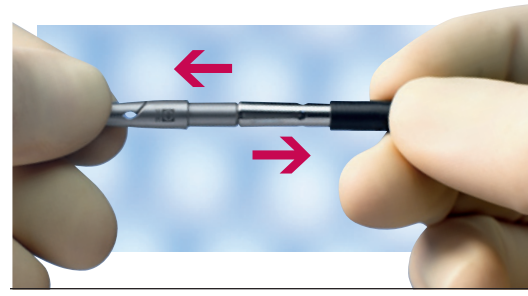


IV Disassemble shaft I
Hold the jaw insert at the tip and remove the outer insulation tube.

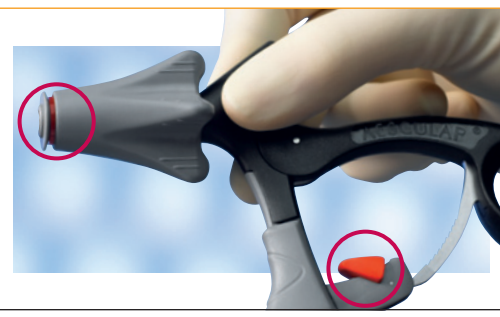


III Prepare handle
If the red ring is not visible, pull the rotation knob back. Deactivate the ratchet by flipping down the red toggle switch.
N.B.: for P0959R only

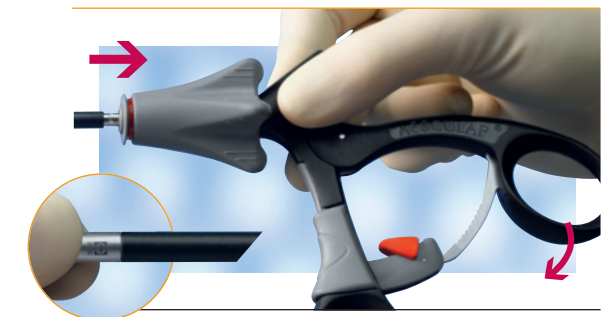

IV Connect shaft and handle
Hold the shaft firmly by its tip and the handle behind the rotation knob. Push the shaft into the handle until it locks in place and the red ring disappears.
Important: Do not hold on to the movable handle component.

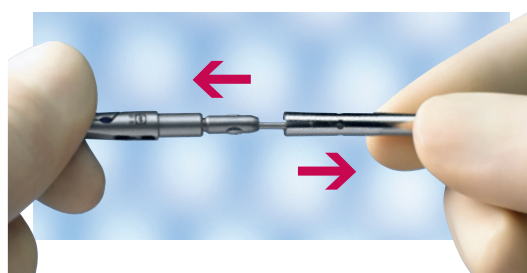


V Disassemble shaft II
Hold the jaw insert at the tip and remove the inner metal tube.

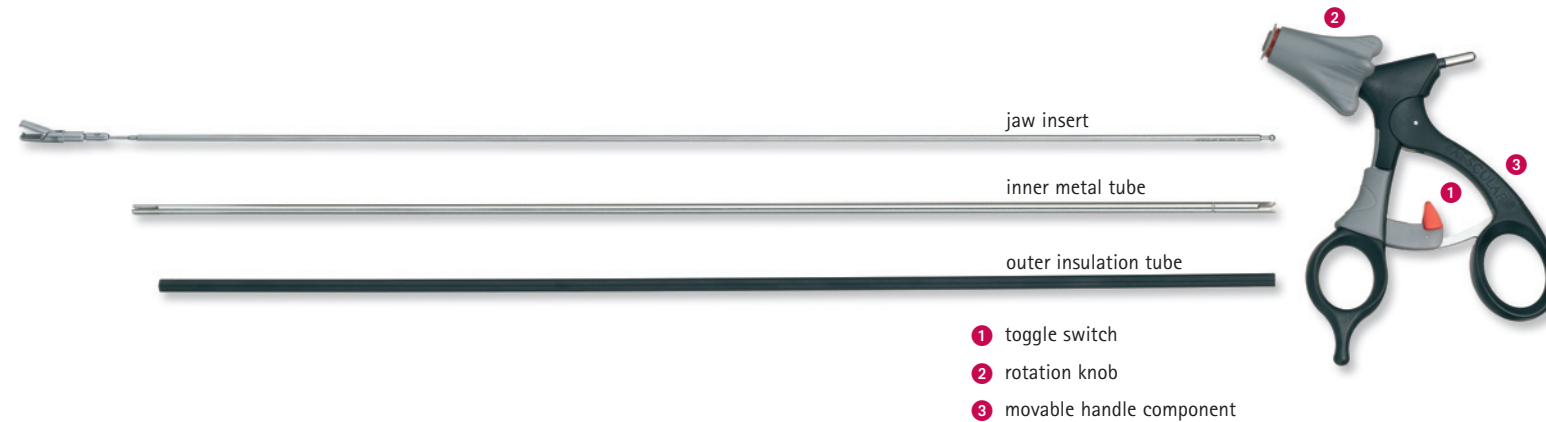

V Function test
Test the assembly of the instrument by opening and closing the handle to ensure the jaw is opening and closing.

CARE

Lubricate the defined places on the instruments (instruments jaw and red toggle switch) with Aesculap Sterilit Oil after each cleaning cycle.

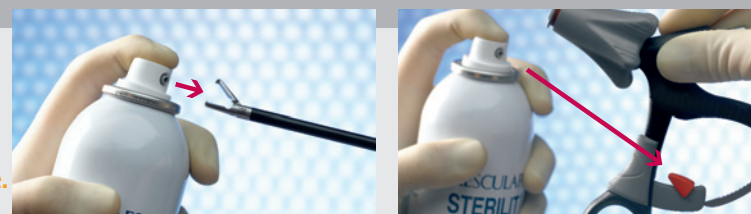

PROCESSING & ACCESSORIES

- Reprocess the product immediately after use
- The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use
- For the latest information on reprocessing, please also refer to the Aesculap Extranet via www.aesculap-extra.net
- For accessories and spare parts, please refer to Brochure C766 or visit www.endoscopy-catalog.com

REPAIRS

In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry out repairs.
Please send products for repair to: Aesculap Technical Service, Am Aesculap-Platz, 78532 Tuttlingen, Germany.
Additional service addresses may be obtained from the same address or via ats@aesculap.de

Aesculap Endoscopic Technology

AdTec® monopolar



AESCULAP®



Disassembly and assembly instructions

B | BRAUN
SHARING EXPERTISE

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.

B | BRAUN
SHARING EXPERTISE

Aesculap AG & Co. KG

Am Aesculap-Platz
78532 Tuttlingen
Germany

Phone +49 7461 95-0
Fax +49 7461 95-2600

www.aesculap.de