



AdTec®bipolar


# Aesculap Endoscopic Technology

## DISASSEMBLY


## ASSEMBLY



**I Disconnect shaft and handle**  
Pull the rotation knob back and hold it in this position.

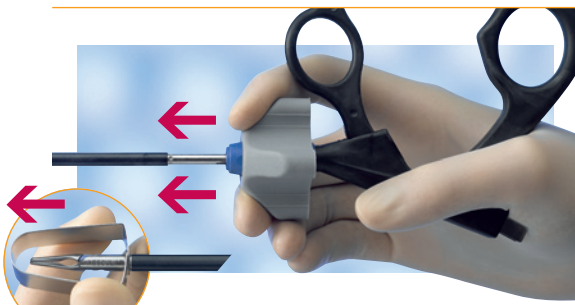

**II Remove shaft from handle**  
Hold the jaw insert firmly by its protecting cap and pull it out of the handle.  
**Important:** Do not hold on to the movable handle component.

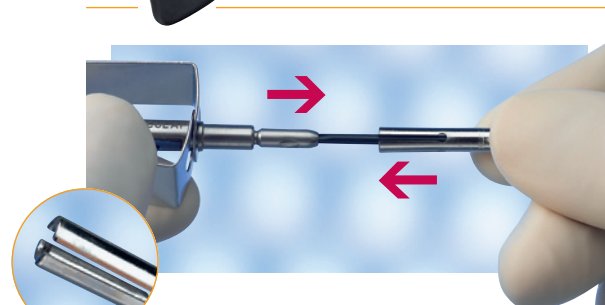


**I Assemble shaft I**  
Slide the metal inner tube onto the jaw insert with its four spring tongues towards the working end until the tongues snap in place.




**II Assemble shaft II**  
Slide the outer insulation tube as far as it will go over the inner metal tube.




**III Disassemble handle**  
Pull the grey rotation knob forwards off the handle.

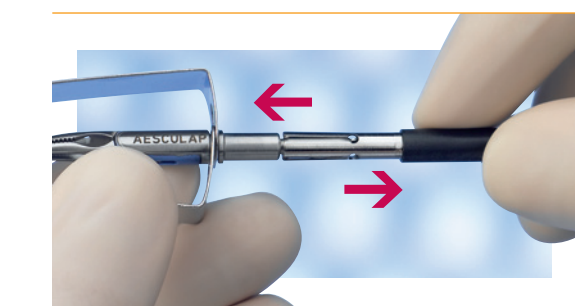


**IV Disassemble shaft I**  
Hold the jaw insert at the protecting cap and remove the outer insulation tube.

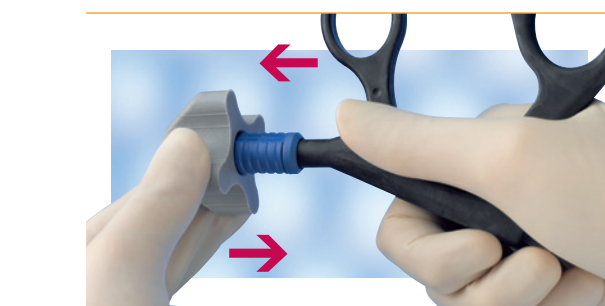


**III Prepare handle**  
Push the grey rotation knob onto the handle from the front. Bring the movable handle component into its forward position.

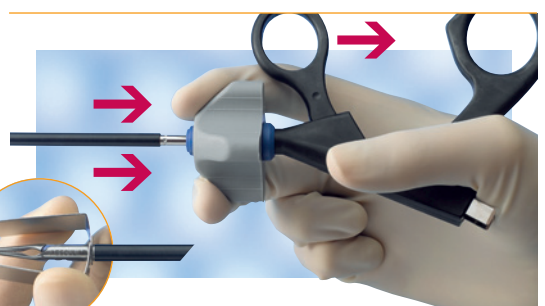

**IV Connect shaft and handle**  
Pull the rotation knob back and hold it. Hold the shaft firmly by its protecting cap and push it into the handle. Release the rotation knob, which moves forward.  
**Important:** Do not hold on to the movable handle component.

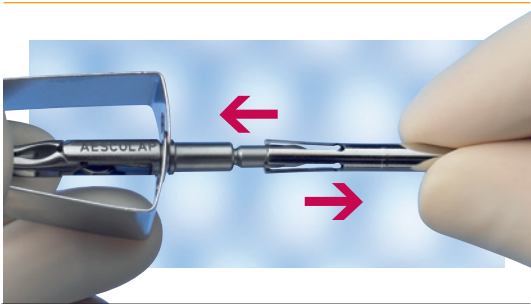

**V Disassemble shaft II**  
Hold the jaw insert at the protecting cap and remove the inner metal tube.

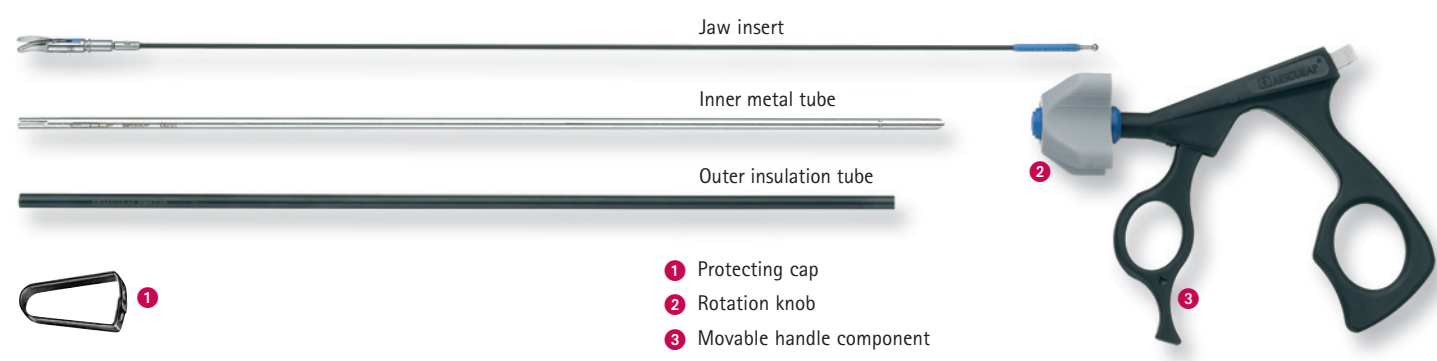

**V Function test**  
Test the assembly of the instrument by opening and closing the handle to ensure the jaw is opening and closing.

## CARE

Lubricate the defined places on the instruments (instruments jaw and red toggle switch) with Aesculap Sterilit Oil after each cleaning cycle.



## STERILE REPROCESSING & ACCESSORIES

- Reprocess the product immediately after use
- The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use. For effective cleaning, particularly of the jaws, we recommend B. Braun Stabimed®.
- For the latest information on reprocessing, please also refer to the Aesculap Extranet via [www.aesculap-extra.net](http://www.aesculap-extra.net)
- For accessories and spare parts, please refer to Brochure C758 or visit [www.endoscopy-catalog.com](http://www.endoscopy-catalog.com)

## REPAIRS

In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry out repairs.  
Please send articles for repair to: Aesculap Technical Service, Am Aesculap-Platz, D-78532 Tuttlingen, Germany.  
Additional service addresses may be obtained from the same address or via [ats@aesculap.de](mailto:ats@aesculap.de)

# Aesculap Endoscopic Technology

## AdTec® bipolar



AESCULAP®



Disassembly and assembly instructions

**B | BRAUN**  
SHARING EXPERTISE

**B | BRAUN**  
SHARING EXPERTISE

Aesculap AG & Co. KG

Am Aesculap-Platz  
78532 Tuttlingen  
Germany

Phone +49 7461 95-0  
Fax +49 7461 95-2600

[www.aesculap.de](http://www.aesculap.de)

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.