


Aesculap® AdTec® mini


Assembly and Disassembly Instructions

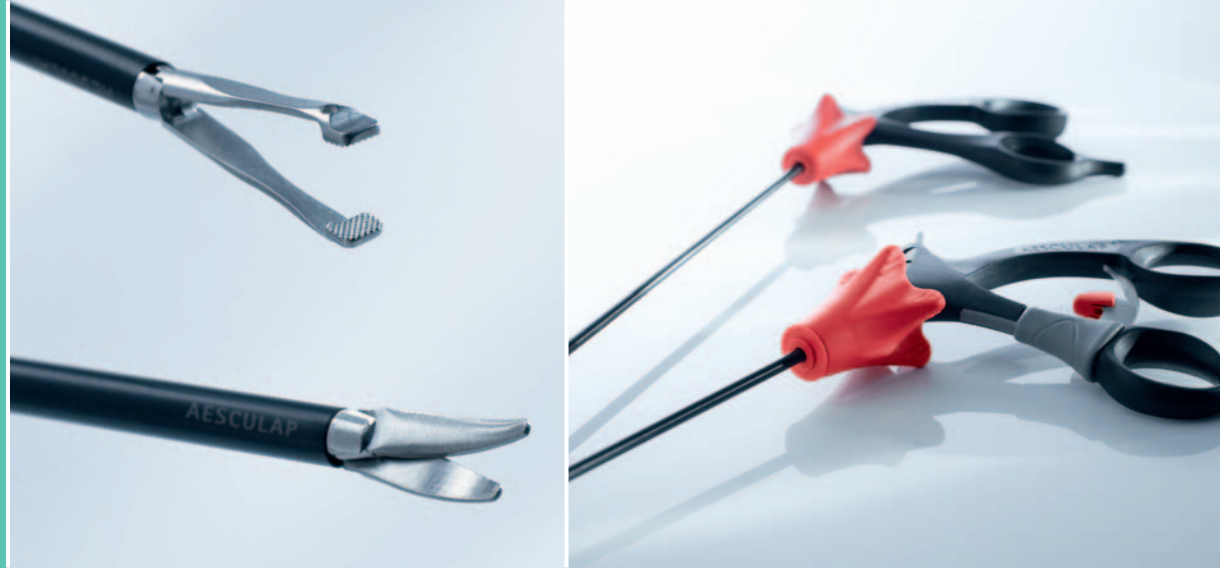

Aesculap Endoscopic Technology




DISASSEMBLY

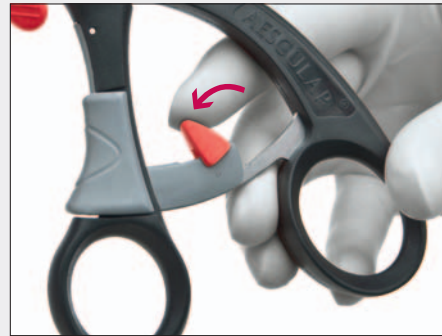


1. Deactivate ratchet on ratchet handle
Flip down red toggle switch.
N.B.: for PO381R only




2. Disconnect shaft and handle
Pull rotation knob back and let movable part of the handle move downwards.

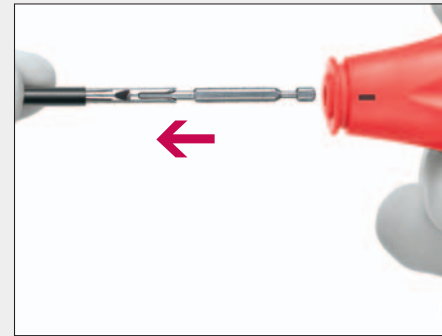


3. Remove shaft from handle
Remove the shaft by pulling the instrument shaft out of the handle.

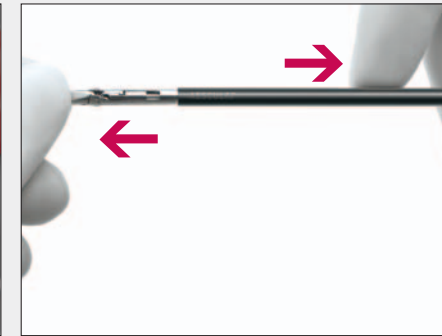

4. Disassemble shaft
Hold the jaw insert at the tip and remove the outer tube.

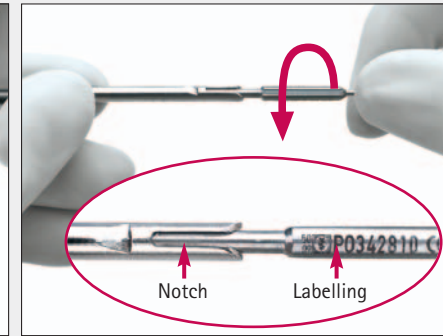


5. Disassemble shaft
Rotate the push rod 90° counterclockwise until the labelling is in line with the notch of the jaw part.

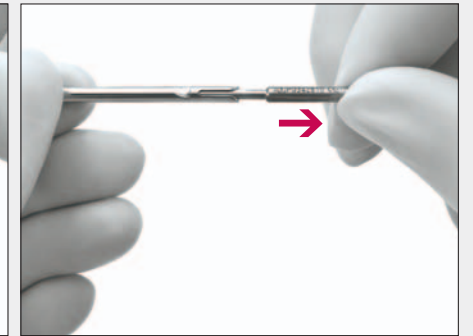


6. Disassemble shaft
Remove the push rod.


ASSEMBLY

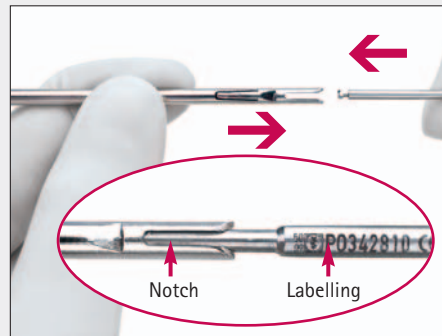

1. Assemble shaft
Slide the push rod into the jaw insert as far as it will go making sure that the labelling on the push rod is in line with the notch of the jaw part.

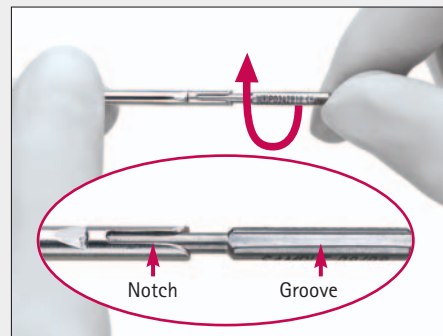


2. Assemble shaft
Rotate the push rod 90° clockwise until it locks into place. The groove is then in line with the notch.

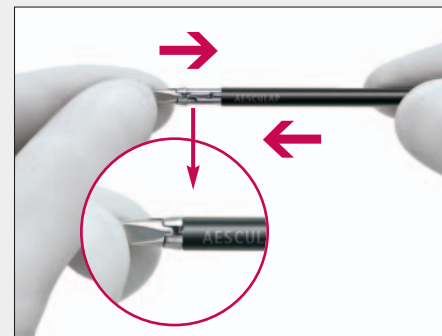

3. Assemble shaft
Slide the black outer tube as far as it will go over the jaw insert.



4. Prepare handle
Deactivate the ratchet by flipping down the red toggle switch.
N.B.: for PO381R only

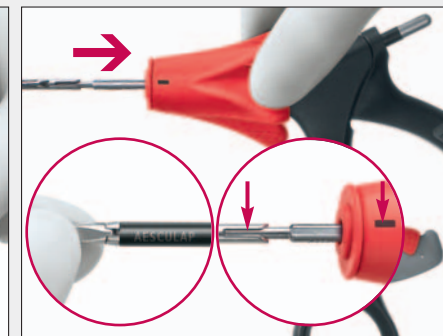

5. Connect shaft and handle
Hold the tip of the shaft and the rotation knob of the handle firmly. Slide the shaft into the handle until it locks into place, making sure that the push rod notch is in line with the mark on the rotating knob.
Important: Do not touch the movable part of the handle.




6. Function test
Test the assembly of the instrument by opening and closing the handle to ensure the jaw is opening and closing.

MAINTENANCE



Lubricate the parts of the instrument specified (jaws and red toggle switch) with Aesculap Sterilit Oil after each cleaning cycle.

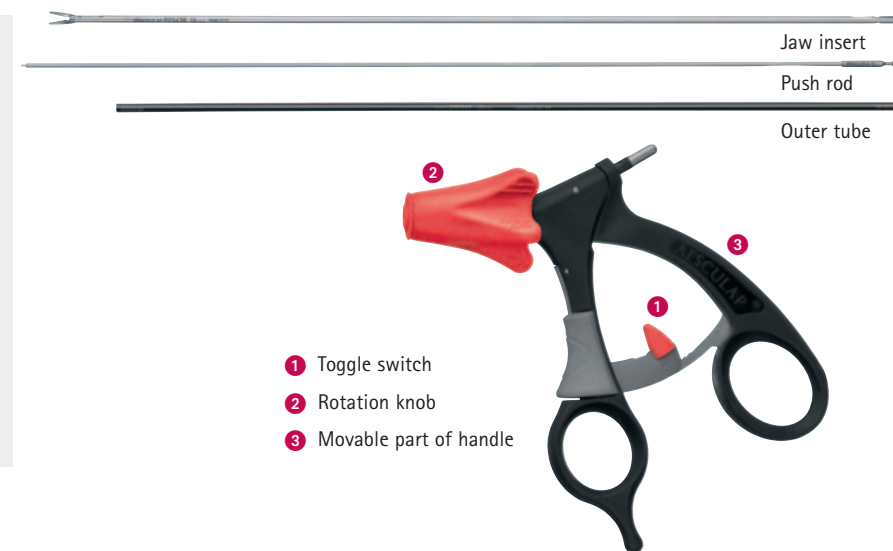

- 1 Toggle switch
- 2 Rotation knob
- 3 Movable part of handle

REPROCESSING & ACCESSORIES REPAIRS

- Reprocess the product immediately after use
- The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use
- For the latest information on reprocessing, please also refer to the Aesculap Extranet via www.aesculap-extra.net
- For accessories and spare parts, please refer to brochure C28011 and C28511 or visit www.endoscopy-catalog.com

In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry out repairs.
Please send products for repair to:
Aesculap Technical Service, Am Aesculap-Platz, 78532 Tuttlingen, Germany.
Additional service addresses may be obtained from the same address or via ats@aesculap.de