

Instruction for use extract of AESCULAP® Caiman in usage and cleaning process

GENERAL USAGE

- Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Avoid the following actions when the Lock is activated:
 - Rotation of the Shaft
 - Articulation of the jaws.
- Risk of insufficient sealing due to contaminated electrode surfaces.
 - Keep the product's contact surfaces clean during surgery. Remove encrusted tissue residues or body fluids with a moistened swab

CLEANING PROCESS

- Clean the instrument if the contact surfaces become encrusted with tissue residue .
- When cleaning the electrodes:
 - Do not activate the HF current.
 - Do not immerse the device jaws in liquid, including Saline solution.
 - Do not clean the device jaws with abrasives.
 - Clean with a moistened swab.
- Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components.

