

# OMNI Competency Assessment

The personal data provided on this form is for assessing staff competency purposes only and will be stored in accordance with the NHS Trust Privacy Policy. No data will be transmitted to B. Braun Medical Ltd or any other third party.

This document is provided by B. Braun and is intended to be used by the trust as a template to assess the competency of staff using the OMNI CRRT device. This template can be tailored to suit the assessment needs of the trust i.e. N/A can be input if the assessment criteria is not relevant. It is the responsibility of the trust to provide training of policies, protocols and guidelines. B. Braun does not provide clinical guidance and all clinical decisions should be made by appropriate medical teams.

This document does not replace the OMNI IFU and additional training material, it should be used once OMNI training has been completed as an additional assessment document and is recommendation only. For full instructions for use, please refer to the OMNI IFU relevant to the current software.

	Direct supervision		Indirect supervision		Competent to practice without supervision
	Trainee initial and date	Assessor initial and date	Trainee initial and date	Assessor initial and date	Assessor name, signature and date
Can correctly set up the filter in CVH					
Can correctly set up the filter in CVHD					
Can correctly set up the filter in CVHDF					
Can provide rationale for use of different fluids with CVH, CVHD, CVHDF and anticoagulant of choice					
Can correctly put the machine into recirculation prior to initiating patient therapy					
Can safely connect the patient to arterial and venous lines in line with infection control protocols					
Can safely commence blood pump and therapy whilst providing rationale regarding haemodynamic monitoring					

# OMNI Competency Assessment

CONFIDENTIAL

	Direct supervision		Indirect supervision		Competent to practice without supervision
	Trainee initial and date	Assessor initial and date	Trainee initial and date	Assessor initial and date	Assessor name, signature and date
Can troubleshoot common alarms					
Can utilise the audio pause and reset alarm icons in response to any alarms on the machine					
Can correctly perform a bag change, ensuring frangible pin is fully broken					
Can correctly perform a syringe change					
Can provide rationale for altering calcium rate in line with protocol (if relevant)					
Can provide rationale for altering citrate rate in line with protocol (if relevant)					
Can correctly calculate calcium ratio (if relevant)					
Can provide rationale for altering dialysate flow in line with protocol					
Can correctly document filter observations in line with trust policy					
Can safely return the blood to the patient to end therapy, temporarily disconnect the patient or change kit					
Can provide rationale for using the treatment options 'end therapy', 'temporarily disconnect patient' and 'change of kit'					