

# Omnican® Lance Soft

LANCETS FOR CAPILLARY BLOOD COLLECTION



## GENTLE PUNCTURE AND IDEAL BLOOD FLOW

A three-facets tip coated with silicone oil means less penetration force for excellent blood flow.




For people with diabetes collecting blood for blood glucose monitoring is a big part of their lives. To make this as comfortable as possible B. Braun offers Omnican® Lance soft. With its thin and precise tip, the lancet ensures an adequate blood flow at minimised pain level. It makes the blood collection with a lancing device more comfortable and supports a successful diabetes therapy.

## STERILITY

Lancet remains sterile as long as protective cap has not been removed.


## VARIED COMPATIBILITY

Rib design with standard dimensions provides compatibility with numerous lancing devices.

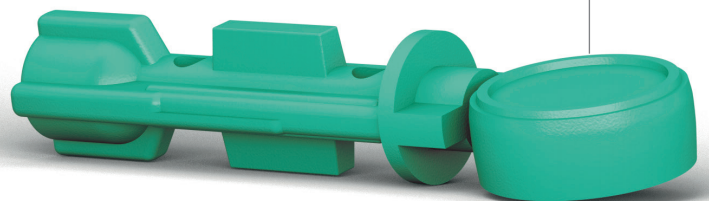

# Omnican® Lance Soft

## HANDLING AT A GLANCE

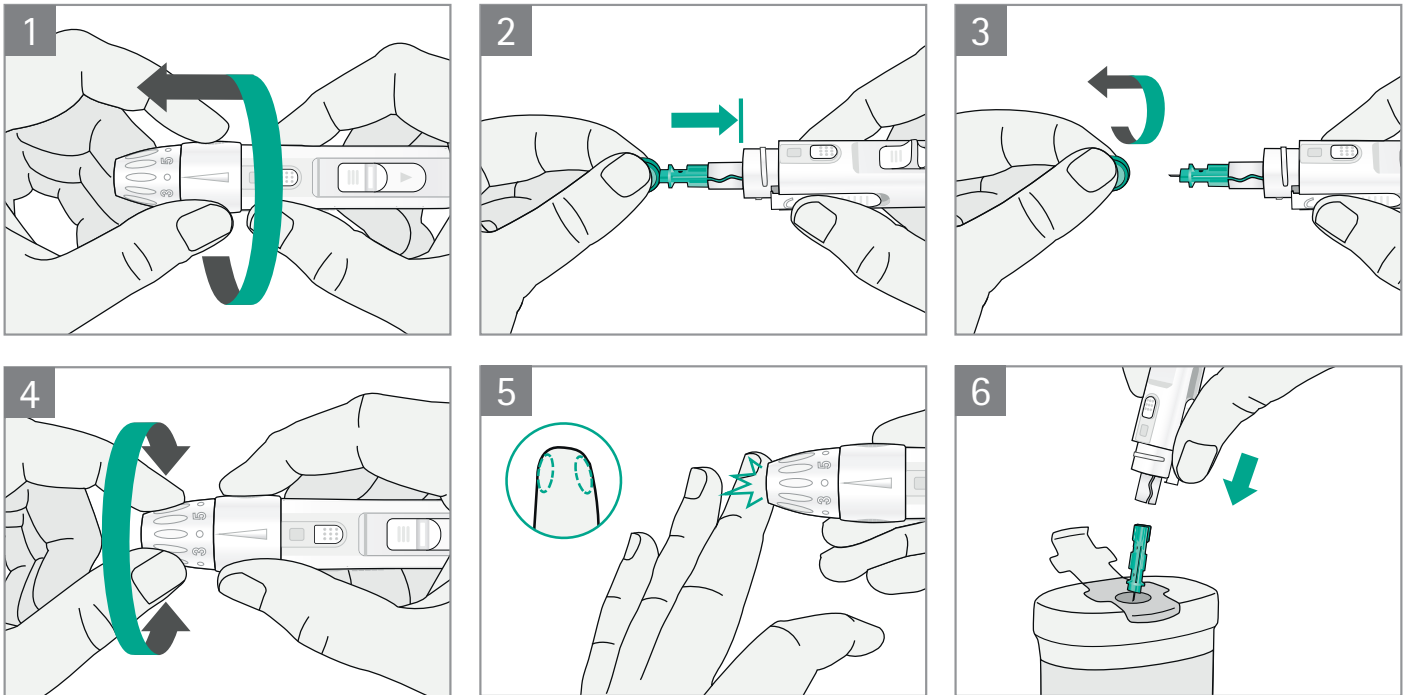

## TIPS AND TRICKS

- Use a new sterile lancet for every blood sample
- Change puncture site systematically using the side of your fingertip
- Avoid index finger and thumb
- Always dispose of the used lancet properly in a suitable container

## VERSATILE COMPATIBILITY

Omnican Lance Soft lancets are compatible with the majority of lancing devices, including but not limited to:

COMPANY	PRODUCT
B. Braun Medical Ltd	Omnican Lance
Bayer	Microlet
GlucRx Ltd	GlucRx Lancets
LifeScan	One Touch Delica
Menarini	Glucoject Plus
Neon Diagnostics	Greenlan
Nipro	TRUEplus

The brands listed are the property of the named companies.

## PRODUCT OVERVIEW

Omnican Lance Soft	Quantity	PIP Code
30 G	200	377-3322

Lancets remain sterile 5 years in accordance with ISO 11137:1995



B. Braun Medical Ltd | OPM | Thorncliffe Park | Sheffield | S35 2PW  
Tel 0114 225 9000 | Fax 0114 225 9111 | [www.bbraun.co.uk](http://www.bbraun.co.uk)

XX-0LSF-07-19