


STANDARD-ANTT CANNULATION PROCEDURE PACK

Instructions For Use

Standard-ANTT:

An aseptic approach that helps protect the **Key-Parts** and **Key-Site** individually, using non-touch technique (NTT) and **Micro Critical Aseptic Fields**.


KEY-PARTS:

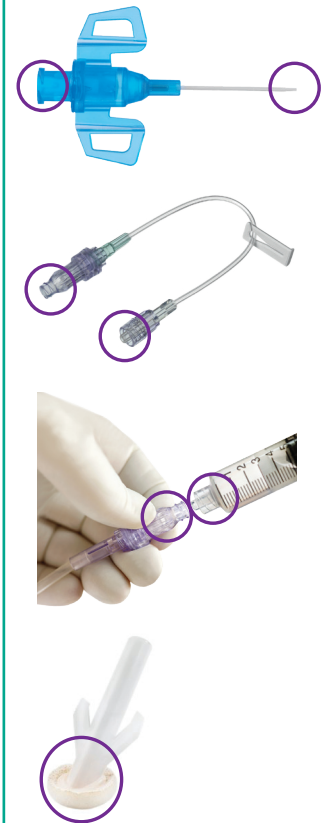

MICRO CRITICAL ASEPTIC FIELDS:



PREPARATION ZONE

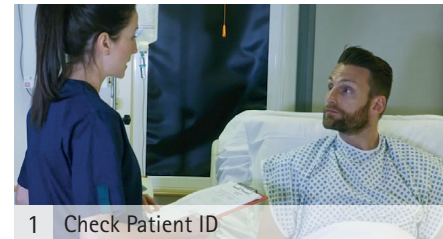

1 Check Patient ID

- Gain consent.
- Undertake initial vein assessment.



2 With clean hands

clean a procedure tray to local policy. Whilst drying..



3 Gather Equipment

Including:
Cannulation pack, PIVC and PPE (including non-sterile gloves) and lay around the tray.



4 Clean hands

with soap and water or alcohol gel.



5 Prepare Equipment

Open pack, lay blue non-sterile drape over tray creating a General Aseptic Field.

PATIENT ZONE



6 Micro Critical Aseptic Fields

Leave Key-Parts in their Micro Critical Aseptic Fields when not being used.



7 Prime Needlefree

Ensure to use NTT to help protect Key-Parts.



8 Key-Part Protection

- Keep the syringe attached to the needlefree valve.
- If running a parallel infusion immediately, clean the second needlefree valve before use.



9 Underarm Drape

Place under patient's arm.



10 Apply Tourniquet


- Palpate and locate vein.
- Loosen tourniquet.

DECONTAMINATION ZONE



16 Clean hands

with soap and water or alcohol gel

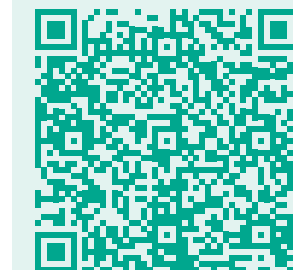

Please scan the QR code with your camera or QR code reader to view an instructions for use video.

Laminated copies for display can be requested by contacting your local B. Braun Medical representative or by emailing:

IVtherapy.bbmuk@bbraun.com stating your 'Trust' and Standard-ANTT Cannulation Procedure Pack 'Product Code'.