

Securing Introcan Safety[®] Deep Access

Shown with Statlock IV Ultra

Correct application of the dressing is essential to ensuring the cannula remains secure.




Cleanse the targeted securement area with a skin wipe to remove any oils or moisturiser.

Allow the area to air dry completely.



To achieve maximum adhesion, apply the Skin protectant Prep Pad over the same area.

Allow the area to air dry completely.

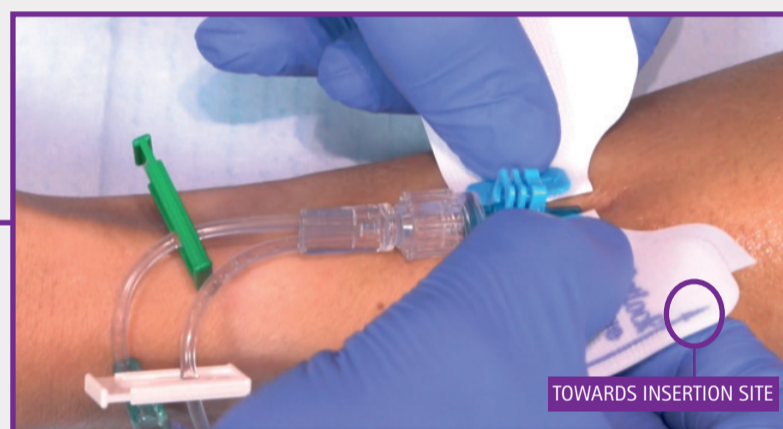

Ensure that the push plate of Introcan Safety Deep Access is facing upwards and the blue arrows of the Statlock orient towards the insertion site, press the retainer over the hub of Introcan Safety Deep Access ensuring it captures the push plate.



Peel away the release liner on both sides to secure Statlock to the patient's skin, covering with a sterile occlusive dressing.

ALWAYS ENSURE CANNULA INSERTION SITE IS NOT OBSCURED

Gauge Size (G)	Length		Catheter ϕ (mm)	Gravity Flow Rate (ml/min)	Power Injection Flow Rate (Contrast Media viscosity = mPa's = 2,3; 27,5)	MPC	NPC
	inch	mm					
 18	2 1/2	64	1.3	85	19; 15	4251620-01	FSP4927
 20	2 1/2	64	1.1	51	13; 8	4251621-01	FSP4926
 22	2 1/2	64	0.9	24	7; 3	4251622-01	FSP4925
 24	1 1/4	32	0.7	17	5; 25	4251623-01	FSP4924