

Introcan Safety® 3 Closed IV Catheter

CLEAR FLASHBACK CHAMBER

UNIVERSAL BACK CUT BEVEL

- Creates a V shaped tricuspid incision for smoother catheter insertion
- Wide choice of insertion angles
- Helps to reduce pain due to lower forces

tricuspid incision

AUTOMATIC SAFETY SHIELD TECHNOLOGY

- Aids in the prevention of needlestick injuries

BLOOD CONTROL SEPTUM

- Minimises exposure to blood
- Reduces clean up time



1 Cleanse area for cannulation with skin cleaning agent as per hospital policy. Ensure the bevel is up.



2 Observe the first flashback in the flashback chamber. Lower the angle of insertion, advance the cannula into the vein slightly to ensure stylet tip is in the centre of the vein.



3 Withdraw stylet and observe second flashback in the catheter shaft.



4 Hold the stylet still and advance cannula smoothly into the vein. Release tourniquet at this point.



5 Stabilise the cannula with the thumb and occlude the vein with the forefinger. Remove the stylet in a straight, smooth action and dispose into a sharps bin. Introcan Safety® 3 has a self-activating safety shield and multi-access blood control septum. The septum helps to prevent back flow of blood however, it is good practice to use vein occlusion technique.



6 Attach a needlefree device and secure cannula with a sterile occlusive dressing as per hospital policy



Please scan the QR code to view the Introcan Safety® 3 IV Catheter clinical demonstration video

For further information please contact your local B. Braun Sales Representative or Clinical Education Specialist.