


# Intrafix® SafeSet

## Reliable and Efficient IV Gravity Set

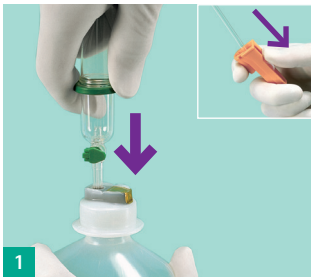

1 Close roller clamp, spike container or bag.




2 Squeeze the drip chamber and fill to the green line. When using rigid containers, open the venting cap.



3 Open the roller clamp and allow the line to prime. Once the line is fully primed close the roller clamp.



4 Remove the green Prime Stop cap.

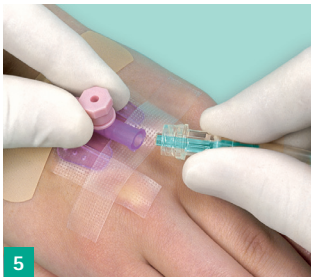

5 Attach the infusion line to the patients vascular access device.



6 Open the roller clamp to the desired flow rate.



### When the container runs empty:

The drip chamber's Air Stop helps prevent air entering the infusion line. Fluid is then maintained at the level of the drip chamber, stopping the infusion line from running dry.

The roller clamp should be closed at the end of the infusion as per your local IV therapy protocol.

For further information please contact your local  
B. Braun Sales Representative or Clinical Education Specialist.