




IryPump® S

Rectal Irrigation – Step by Step Guide



Turn on the IryPump® S to check if the battery is properly charged.

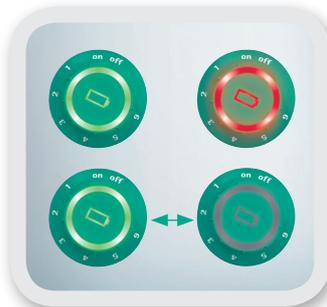

Ring is red = IryPump® S needs charging. Ring is green = ok to begin irrigation.



Remove the water container from the pump module by pressing the side release button.



Extend the water container. Pull the inner section up until it audibly clicks into position.



Fill the water container with the advised amount of water. Temperature between 36°C and 38°C.



Attach the filled water container back onto the pump module.



Use the quick connector to fasten the tubing system onto the pump.



Connect the other end of the tubing system to the cone.



Expel all air from the tube by turning the control wheel to level 3.



Ensure that you are in a comfortable position. Lubricate the cone and carefully insert it into the anus.



Use a water based lubricant to lubricate the cone.



Hold the cone in place by applying a slight pressure throughout the irrigation procedure.



Switch the IryPump® S on, making sure you keep the cone in place.



Set the pump to the level suggested by your nurse. As the water volume decreases this shows the water is flowing correctly.



A pre-rinse using 150 ml – 250 ml of water can help achieve a more effective result.



Once all the water has flowed in, turn the IryPump® S off and wait 2 to 3 minutes before removing the cone.



After a few minutes the defecation process will begin, the time will vary from person to person.



Once the irrigation process has finished. Clean your IryPump® S System with warm soapy water, then dry it and recharge if necessary.

IryPump® S Set	Quantity	REF
Complete starter set, contains	1 set	29120E
<ul style="list-style-type: none"> ▪ IryPump® S (electrical pump, water container and power supply) ▪ IryCone®+ (cone and tubing) ▪ IrySleeve® Irrigation Sleeves 60 mm ▪ Iryflex® Faceplate ▪ B. Braun StomaCare Belt ▪ Iryfix® ▪ B. Braun Petite ▪ Softima® Active Mini ▪ Carrying Case 		
		
IryPump® S Accessories	Quantity	REF
<ul style="list-style-type: none"> ▪ IryCone®+ (cone and tubing) ▪ IryPump® Water container 	1 piece	29220E
	1 piece	29240



Please call our freephone careline for more information
 0800 163 007 (Ireland 1800 409 538) or visit our website:
www.bbraun-emerald.co.uk