# Prontosan<sup>®</sup> – Wound Irrigation Solution

<b>i</b>	Vor Anwendung Gebrauchsinformation lesen Read instruction for use. Прочетете указанията за употреба. Přečtěte si pokyny k použití Láes brugsanvisningen. Lugege kasutusjuhendit.
 8w	Haltbarkeit nach Öffnung: 8 Wochen Shelf life after opening: 8 weeks Срок на годност след отваряне: 8 седмици. Дoba použitelnosti po otevření: 8 týdnů Holdbarhed efter åbning: 8 uger. Säilivusaeg pärast avmist: 8 nädalat.
	Verwendbar bis: Monat und Jahr Use by: month and year Да се използва до: месец и година Spotřebujte do: měsíc a rok Anvendes senest: måned og år. Säilivusaeg: ku ja aasta
<b>REF</b>	Katalognummer Catalog number Каталоген номер Каталогové číslo Varenummer Katalogi number
<b>LOT</b>	Chargennummer Batch number Партиден номер Číslo šarže Varenr. Partii number
	Darf nicht in die Hände von Kindern gelangen. Keep out of reach of children. Да се сьхранява на място недостъпно за деца. Учювавайте mimo dosah dětí. Opbevares utilgængeligt for børn. Hoida lasteile kättesaamatus kohas.
	Hersteller Manufacturer Производител Výrobce Producent Tootja
	A-sterilisiert sterile A стерилизовано с A steriilni A steriil A sterilie A
	Nur zum einmaligen Gebrauch. Single use only. Само за еднократна употреба. Pouze na jedno použití. Kun til engangsbrug. Ainult ühekordseks kasutamiseks.

## GB Prontosan<sup>®</sup> – Wound Irrigation Solution

**Instructions for use for cleansing, rinsing and moistening of acute, chronic and infected skin wounds, 1st and 2nd degree burns**

**1. Introduction:** Chronic skin wounds are often coated with slough, necrotic tissue and/or biofilm. These coatings are difficult to remove and lead to delayed wound healing. Prontosan<sup>®</sup> Wound Irrigation Solution is able to remove these barriers to wound healing through its cleansing activity.

Acute wounds also require proper cleansing as they are generally contaminated with debris and microorganisms. These contaminants can interfere with the normal wound healing process and lead to complications such as infection. Due to the unique combination of ingredients (i.e. the antimicrobial substance, polihexanide, and the surfactant, be-taine), Prontosan<sup>®</sup> is ideal for the prevention of biofilm formation.

#### 2. Product profile and areas for use

For cleansing and irrigation plus moistening of:

a) acute non-infected and infected wounds - traumatic wounds (e.g. skin lacerations, bites, cuts or crush injuries); -postoperative wounds.

b) chronic non-infected and infected wounds including: pressure sores; venous ulcers; diabetic ulcers.

c) thermal and chemical wounds: 1st and 2nd degree burns; chemical burns.

d) post-radiation wounds.

e) fistulas and abscesses.

f) entry ports of urological catheters, PEG/PEJ tubes or drainage tubes.

g) peristomal skin.

For intraoperative cleansing and irrigation of wounds.

For instillation in combination with negative pressure wound therapy.

For moistening encrusted dressings and bandages prior to removal.

**3. General use:** Prontosan<sup>®</sup> Wound Irrigation Solution (see separate product information) should be used for wound cleansing prior to treatment with Prontosan<sup>®</sup> Wound Gel or Prontosan<sup>®</sup> Wound Gel X.

Prontosan<sup>®</sup> Wound Irrigation Solution may be used to saturate compresses, gauzes or pads for cleansing as needed. Application should be conducted frequently in order to achieve and maintain an visually clean wound. Prontosan<sup>®</sup> Wound Irrigation Solution may be warmed to body temperature prior to use.

Wound dressings and bandages are frequently encrusted and adhere to the wound surface. Removal of these dressings can traumatise the wound bed, which in turn delays wound healing. In cases where dressings and bandages are difficult to release due to encrustation, soaking the dressing with Prontosan<sup>®</sup> Wound Irrigation Solution is recommended until they can be gently removed.

**4. Wound cleansing:** The entire wound, as well as the surrounding area, should be thoroughly cleansed with Prontosan<sup>®</sup> Wound Irrigation Solution. For non-severe and acute wounds rinsing is sufficient. For wounds covering a large area and also wounds difficult to access, soaking the affected area with Prontosan<sup>®</sup> Wound Irrigation Solution for at least 15 minutes with a saturated compress is recommended. The solution should be used undiluted

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